



MJM International Corporation

“We Do Care – We Do More”

632 S. Jackson ▪ Edinburg, TX 78539 ▪ P.O. Box 720054 ▪ McAllen, Texas 78504

1-800-270-6199 ▪ 956-781-5000 ▪ Fax: 956-781-5552 ▪ www.mjminternational.com ▪ sales@mjminternational.com

700 SERIES RECREATIONAL CHAIRS

**722-ATC-YEL-KD, 722-ATC-ELR-YEL-KD,
722-ATC-ELR-REC-YEL-KD, E720-ATC-YEL-KD,
725-ATC-YEL-KD and 750-ATC-YEL-KD**



Carefully Review This Manual Before Using Any Of Our Recreational Chairs:

⚠ WARNINGS

- Individual should never be left unattended in our Recreational Chairs.
- The equipment should always be used under proper supervision.
- Assistance should always be provided to the individual when transferring in and out of the equipment.
- Equipment may not be appropriate for all individuals. Assessment should be conducted by a skilled caregiver for proper suitability for the individual using the equipment.
- Always abide by weight limit capacities for each product.
- Never allow the user to lean outside the frame of the equipment.
- Never allow the user to suddenly shift weight in any way creating a tipping hazard for the user and the equipment.
- Always use the equipment's safety belt and/or shoulder harness if ordered separately.

- The improper use of this equipment or failure to comply with all directions and warnings may result in serious injury.
- Before use ensure the equipment has been properly assembled according to the assembly instructions provided.
- Do not use this equipment on an incline, always make sure all wheels are in constant contact with the ground surface.
- Always store equipment in a dry ventilated environment.
- Before every use, ensure wheels are all functioning properly and secured to the equipment.
- **DO NOT USE** any equipment if the following is evident:
 - Equipment appears unsteady
 - The wheels fail to rotate with ease
 - The fabric appears torn or sagging
 - Cracks are observed in the fittings or frame
 - The equipment appears to be compromised in any way, shape or form
- Do not attempt to modify the equipment, use only the wheels and accessories provided for the equipment. Any use of third-party wheels and/or accessories may pose a hazard to the user and will void the manufacture warranty.

Proper Care & Maintenance

- **PVC Piping**
 - ✓ Wipe down piping with non-abrasive cleaner.
 - ✓ **DO NOT** use solvents.
 - ✓ Restore luster with spray wax.
- **Mesh Fabric (Sling)**
 - ✓ If laundering is required, use only **COLD** water in large machine with regular detergent and one-cup bleach.
 - ✓ **DO NOT** use softeners or solvents.
 - ✓ Spin-dry only; machine heat will shrink mesh fabric.
 - ✓ Hang damp fabric on frame and allow to air dry, returning to its original shape.
- **Footrest and Drop Arms**
 - ✓ Remove build-up from sliding surface with a soft bristle brush and soapy solution.
 - ✓ Lubricate sliding mechanism with furniture spray wax to restore ease in movement.
 - ✓ When lifting drop-arm, use two hands, one at each end of the section. Lifting only at hand grip will put undue pressure against the sliding fitting.

Recommendations:

- **Avoid areas of:**
 - Loose sand entrances-- loose sand will hinder and stop the mobility of the chair. An occupied Recreational Chair will sink in loose sand and the wheels will not roll properly due to the weight of the chair and an individual on board.
 - Gravel—The Recreational Chair will sink in the gravel due to the weight of the chair and an individual on board; the wheels will not roll properly in gravel.
- **For better mobility use on:**
 - Smooth packed sand
 - Flat dirt
 - Grass

- Cement
- Boardwalk ramps
- Walk-ways
- Designated handicap beach access entrances

Frequently asked questions:

- **Is this Recreational Chair motorized?**
 - No, physical force is required to set the chair in motion and continue mobility of the chair.
- **Can turns be made on Recreational Chairs?**
 - Yes, models 722-ATC-YEL-KD and 722-ATC-ELR-YEL-KD have 2 rigid wheels and 2 swivel wheels which allow this chair to make sharp turns. Model E720-ATC-YEL-KD will need a wide space to make turns.
- **Is the umbrella removable?**
 - Yes, the umbrella can be easily removed by removing clamp on the backrest and pulling the umbrella from the pvc pole upwards.
- **Can these Recreational Chairs be submerged in water?**
 - No, submerging these Recreational Chairs in water could cause the chair to destabilize making this a hazardous situation.
- **Can these products be disassembled?**
 - Yes, we offer knock-down versions of our Recreational Chairs that disassemble and allow for easy storage and transferring.



722-ATC-YEL-KD

(Shown Stored In A Mid-Size SUV)



E720-ATC-YEL-KD

(Shown Stored In A Small Compact Size Car)

**Recreational Chairs Shown In The Knockdown Position
Allows For Easy Storage and Transferring**

THE GLOBAL LEADER IN PVC MEDICAL EQUIPMENT

323 East Owassa, Suite 4, Edinburg, TX 78539 1-800-270-6199 / 956-781-5000 / Fax 956-781-5552

www.mjminternational.com hmoroles@yahoo.com